

Customer Information Bulletin

Subject:

NextDent® 5100

NextDent® Model 2.0 White: tips to make white prints their whitest

Issue:


Unless certain, specific precautions are taken, Model 2.0 prints can come out not as white as desired.



Solution:

Use the following guidelines to ensure your parts come out as white as possible:

1. If you have used the Resin Tray for another material besides Model 2.0 White, ensure you thoroughly clean the Resin Tray before using Model 2.0 White inside. Ensure you clean the area around the gasket (blue border seen on the right), as resin can get trapped inside.

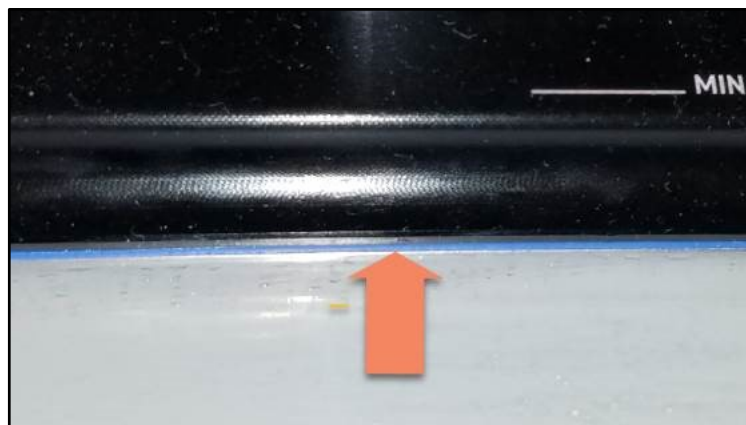

2. When cleaning parts printed with Model 2.0 White, use fresh >90% alcohol solvent for BOTH part baths.
3. Ensure the print platform has been cleaned thoroughly in between uses, so as not to contaminate the Resin Tray with a different resin.